

ISTRUZIONI PER L'USO

BENDE ADESIVE NON STERILI

NOME COMMERCIALE DEL PRODOTTO: PAP02NBSC

FABBRICANTE: RAYS S.p.A. – Via Francesco Crispi 26 – 60027 – Osimo (AN)

CONSERVAZIONE:

Conservare in luogo fresco ed asciutto lontano da fonti di calore.

INDICAZIONE MONOUSO: Dispositivo medico monouso

DESTINAZIONE D'USO:

Dispositivo medico per medicazioni.

ISTRUZIONI PER L'USO:

Lavare accuratamente le mani

Aprire la confezione.

Adattare la parte adesiva alla pelle pulita e asciutta.

Rinnovare la medicazione all'occorrenza..

Non applicare direttamente sulla cute lesa.

In caso di riutilizzo è possibile l'instaurarsi di infezioni: consultare un medico.

LATEX FREE:

Il dispositivo è privo di lattice per evitare il rischio di reazioni allergiche.

SMALTIMENTO:

Lo smaltimento deve essere effettuato in ottemperanza alle norme nazionali vigenti. L'incenerimento o lo smaltimento in discarica devono essere svolti in condizioni controllate. Il prodotto che non è venuto a contatto con agenti infettivi o altri materiali rischiosi, non può essere considerato rifiuto pericoloso.

Le suddette Istruzioni per l'Uso devono essere conservate dall'utilizzatore per tutta la durata del DM.



Health&Safety

RAYS SPA

via Francesco Crispi - 60027 Osimo (AN)

Tel. 071 2868935 - Fax 071 2868911 - www.rays.it - info@rays.it

INSTRUCTIONS FOR USE

NON-STERILE ADHESIVE PLASTERS

PRODUCT TRADE NAME: PAP02NBSC

MANUFACTURER: RAYS S.p.A. – Via Francesco Crispi 26 – 60027 – Osimo (AN)

STORAGE:

Store in a cool, dry place away from heat sources.

SINGLE USE INDICATION: Single use medical device

INTENDED USE:

Medical device for dressings.

INSTRUCTIONS FOR USE:

Wash your hands thoroughly

Open the package.

Adapt the adhesive part to clean, dry skin.

Renew the dressing if necessary.

Do not apply directly to damaged skin.

In case of reuse, infections may develop: consult a doctor.

LATEX FREE:

The device is latex-free to avoid the risk of allergic reactions.

DISPOSAL:

Disposal must be carried out in compliance with current national regulations.

Incineration or landfill disposal must be carried out under controlled conditions. The product that has not meet infectious agents or other risky materials cannot be considered hazardous waste.

These Instructions for Use note must be preserved by the user for the duration of the DM.



Health&Safety

RAYS SPA

via Francesco Crispi - 60027 Osimo (AN)

Tel. 071 2868935 - Fax 071 2868911 - www.rays.it - info@rays.it