GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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JIANGSU YUYUE MEDICAL EQUIPMENT & SUPPLY CO.,LTD.

No.1 Baisheng Road Development Zone, Danyang,

Jiangsu 212300 CHINA

www.yuwell.com

Made in China

H027B

Metrax GmbH

Rheinwaldstr. 22, 78628 Rottweil, GERMANY

Gima S.p.A.

Via Marconi, 1

20060 Gessate (Mi) - Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

Gima 42914

Commode Chair Carrozzina da comoda Chaise percée Silla Inodoro Cadeira sanitária Toilettenstuhl Καρέκλα Τουαλέτας Krzeseł sanitarne

User's Manual - Manuale d'uso - Manuel de l'utilisateur - Manual del usuario
Manual do utilizador - Benutzerhandbuch - Εγχειρίδιο χρήσης
Instrukcja użytkownika

Please read the user's manual closely before using!
Leggere attentamente il manuale d'uso prima dell'utilizzo!
Merci de lire cette notice attentivement avant d'utiliser le produit !
¡Lea detenidamente este manual de usuario antes del uso!
Leia atentamente o manual do utilizador antes de utilizar!
Bitte lesen Sie vor der Verwendung sorgfältig die Bedienungsanleitung!
Διαβάστε προσεκτικά το εγχειρίδιο χρήσης πριν από τη χρήση!
Przed użyciem dokładnie przeczytać instrukcję użytkownika!

ENGLISH

I. Product usage

II. Maintenance

This product issuitablefor disable people.

Contraindications: None

III. Main properties

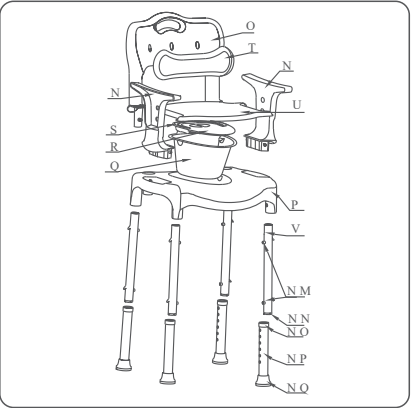
H027B commode chair type: removable armrest and backrest, and height adjustable in six levels.

H027B commode chair has a gray main body and adopts foot pads.

IV. Basic structure

It is composed of a commode rack, telescopic tubes, an armrest, an armrest tube, a backrest tube, positioning pins, foot pads, a commode frame, a commode lid, a commode bucket, and a commode bucket lid.

The basic structure is as shown in the figure

1. Armrest (including the structure functions of armrest and armrest tube)
2. Backrest (including the structure functions of backrest and backrest tube)
3. Chair seat (including the structure functions of commode rack and commode frame)
4. Commode bucket V.
5. Commode bucket lid
6. Commode bucket handle
7. Backrest cushion
8. Seat cushion (including the structure function of commode lid)

9. Upper leg tube
10. Positioning pin
11. Inner tube plug
12. Pipe sleeve
13. Telescopic tube
14. Foot pads

V Usage

Selection of medical walking aids: H027B commode chair have removable armrests and backrests, and the height is adjustable among six levels. You can choose a product type according to your needs.

Height adjustment: Press the protruding head of the positioning pin (10) with one hand, and hold the telescopic tube (13) with the other hand, move it up and down to adjust to a suitable position, and ensure the protruding heads of positioning pins (10) of the 4 feet are fastened firmly to the holes of the same height to avoid danger.

Installation and use of H027B commode chair:

- ① Insert 4 upper leg tubes (9) into the foot holes of the seat (3) until hear a “click” sound. When the protruding head of the pin (10) pops out of the positioning hole, it means the leg tubes are installed.
- ② Insert the armrest (1) and the backrest (2) into the fixing holes of the seat (3) in the same way of ① until hear a “click” sound. When the protruding head of the pin single pin pops out of the positioning hole, it means the leg tubes are installed.
- ③ Place the commode bucket components (4)(5)(6) into the middle hole of the seat (3), and then put on the seat cushion (8) in place.
- ④ Clip the backrest cushion (7) into the 3 small oval holes on the backrest (2). Now the installation is complete and it can be used normally.
- ⑤ When using the commode function, please remove the seat cushion (8) and the commode lid (5), and cover it again after using to prevent the bad smell of stool and urine; when using the bath function, it is recommended to remove the commode bucket components (4)(5)(6).

Removal of the armrest and backrest: Two people are needed. One person presses down the armrest (1) and the pin protruding heads at the lower part of the backrest (2) with both hands, and the other person holds and lift the armrest (1) and backrest (2) lightly to remove them.

V. Maintenance

Symbol and signification

Symbol	Implication	Symbol	Implication
	Fragile		Keep up
	Keep dry		Recyclable
	Manufacturer		Date of manufacture
	EC-Representative		Serial number
	Medical device		See instruction for use
	CE marking and notify body number		Product code
	Unique device identifier		Imported by

Before use, please check whether all fittings of the product are fastened firmly and make sure there is no looseness.

Do not use the product on sloped roads, and be careful on smooth roads.

When adjusting the height, make sure that the height of the four feet is the same, and the positioning pins (10) are fastened firmly.

When installing the armrest (1), backrest (2) and upper leg tubes (9), make sure there is a “click” sound and check 05 06

Symbol and signification whether they are fastened firmly, so as to ensure safety. The foot pads (14) of H027B commode chair are made of plastic, and should be replaced in time if any wear to avoid accidents.

Attention: People with lower limbs inconvenience must pay attention to safety when using this product.

- Transportation: It should be handled with care during transportation.
- Do not throw, press it with heavy load or roll it over.
- Storage: It should be placed in a dry, ventilated and stable room with no corrosive materials.
- Use a soft dry cloth for wiping the medical walking aid.
- Discard the waste in accordance with the relevant local environmental protection regulations.
- For devices intended for use by lay persons, the circumstances in which the user should consult a healthcare professional.

All specifications and product configurations are subject to change without notification.

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