**BARELLA DI SOCCORSO
EMERGENCY STRETCHER
BRANCARD D'URGENCE
NOTFALLTRAGE
CAMILLA PARA EMERGENCIAS
MACA DE EMERGÊNCIA
ΦΟΡΕΙΟ ΕΚΤΑΚΤΗΣ ΑΝΑΓΚΗΣ
АВАРИЙНА НОСИЛКА
POHOŤOVOSTNÍ NOSÍTKA
NØDBÅRE
KANDERAAM HÄDAOLUKORRAKS
HÄTÄPAARIT
NOSILA ZA HITNE SLUČAJEVE
VÉSZHELYZETI HORDÁGY
NEŠTUVAI SKUBIAI PAGALBAI
AVĀRIJAS NESTUVES
FØRSTEHJELPSBÅRE
NOODBRANCARD
NOSZE RATUNKOWE
TARGÅ DE URGENTÅ
NÚDZOVÉ NOSIDLÁ
NOSILA ZA NUJNE PRIMERE
NØDBÅR**

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GIMA 56035

Zhangjiagang Xiehe Medical Apparatus & Instruments Co., Ltd
No.7th, Middle Xinzha Road, Zhashang Industrial Zone,
Yangshe Town, Zhangjiagang City, Jiangsu 215600, China
Made in China

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Name: Emergency Stretcher YXH-4D

Accessories:

three safety belts metal lock & polyester fiber.

Technical data:

1. Product size (LxW xH): 160x44x7cm, the longest can be up to 210cm.
2. Folding size (LxW xH): 120 x44 x7cm
3. Carton size: 165x48x8cm (1pc/ctn)
4. N.W.(1pc): 8kg G.W.: 11kg
5. Load bearing: ≤159kg

Description:

1. This is the new product among the series of the scoop stretcher, it is firm and light, easy to clean and operate. It is the ideal product not only to the patients but also to the operators.
2. Twin safety lock for smooth and unbinding locking and unlocking.
3. Head section recessed to maintain patient's cervical alignment.
4. Different length options according to the height of patients.
5. The thermally-treated polymer material will not make the patients too hot or too cold, the surface is easy to clean and impervious to fluids.

Intended use:

It is used to help the transportation of the injured people and for the rescue, immobilization and transport of the wounded".

Attention:

When carrying patients with this stretcher, be sure fasten the safety belts for safety sake.

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Maintenance:

1. Keep clean as routine.
2. Often check if any parts loose or not.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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