

Adult/Pediatric Airway Adapter A3

It is only applicable to the Capno-T produced by Kingst.
Instruction for use:

1. For monitoring the adult or pediatric patients with endotracheal tube size greater than 4.0mm.

2. Press the airway adapter onto the Capno-T, it will 'click' into place when properly seated.

3. Perform Zeroing when replacing new adapter(See manual for details).

4. Install the Capno-T with airway adapter to the ventilator/ anesthetic circuit or oxygen delivery device.

5. DO NOT place the adapter between the endotracheal tube and an elbow to avoid the adapter windows blocked by patient secretions.*

REF 668502

LOT XXXXXXXX



XXXX-XX

XXXX-XX


Beijing Kingst Commercial & Trade Co., Ltd.
The 6th floor of No.2-7,Road Beibinhe,
Xibianmen, Dist xicheng,Beijing,P.R.C

EC REP SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands


* For more information please see the User Manual

Ver: V1.0

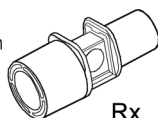

Rx
Only

Single
Patient
Use Only

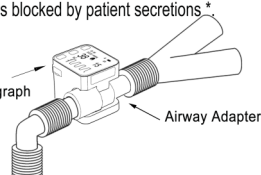

Adattatore per le vie aeree A3 per pazienti adulti/pediatrici

È unicamente applicabile al Capno-T prodotto da Kingst.
Istruzioni per l'uso:

1. Per il monitoraggio di pazienti adulti o pediatrici con dimensioni del tubo endotracheale superiori a 4,0 mm.

2. Premere l'adattatore per le vie aeree sul Capno-T; un clic segnerà l'inserimento corretto.

3. Quando si effettua la sostituzione con un nuovo adattatore, effettuare l'azzeramento (v. il manuale per i dettagli).

4. Installare il Capno-T con l'adattatore per le vie aeree al ventilatore/circuito anestetico o al dispositivo di erogazione dell'ossigeno.

5. NON posizionare l'adattatore tra il tubo endotracheale e un gomito per evitare che le finestre dell'adattatore siano bloccate dalle secrezioni dei pazienti*

REF 668502

LOT XXXXXXXX



XXXX-XX

XXXX-XX


Beijing Kingst Commercial & Trade Co., Ltd.
The 6th floor of No.2-7,Road Beibinhe,
Xibianmen, Dist xicheng,Beijing,P.R.C

EC REP SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

* Per maggiori informazioni, consultare il manuale dell'utente

Ver: V1.0

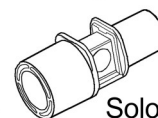

Solo
Rx

Esclusivamente
per uso
monopaziente



Importato da / Imported by:

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

MADE IN CHINA



6

941900

605824



000004