90.00mm

Value&Safe

Filtering Mask - Earloop - black
Masque filtrant
Avec élastique - noir
Mascherina filtrante
Con elastico - nera
Mascarilla filtrante
Con elástico - negra
Máscara de filtragem
Com elástico - preta

FFP2 NR
>94%

20
Piecs
Pièces
Pezzi
Piezas
Peças

🇬🇧 This respiratory mask is suitable for protection against solid aerosols and non-toxic liquids - **NON-STERILE, SINGLE-USE, DUSTPROOF** - 42,8% nonwoven fabric, 28,6% hot-air cotton, 28,6% meltblown fabric - Do not press excessively during transport, in order to avoid damaging the package. 3-year validity, do not use beyond expiration date.

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🇬🇧 Wearing instructions

1. Place the nose piece on your nose and the mask on your chin.
2. Pull the elastic loops behind your ears and adjust the mask.
3. Adjust the nose piece to your nose.
4. Make sure that the mask fits close to your face.

⚠️ **WARNING**

- a. Risk may be expected if the product is used in a surgical setting or in a setting with significant exposure to bodily or other hazardous liquids or fluids; or used in a clinical setting where the risk of infection through inhalation is high; or used in the presence of a high intensity heat source or flammable gas.
 - b. The product is not designed for use in an anoxic or an underwater environment.
 - c. Check for integrity before use.
 - d. This product is not a respirator.
 - e. The mask does not eliminate the risk of contracting any disease or infection.
 - f. Failure to properly use and maintain this product could result in illness.
 - g. Do not use beyond the period of validity.
 - h. Do not use if the packaging is damaged.
 - i. Do not use with oxygen and only in a ventilated area with > 19% oxygen and for one work shift. Any modifications, damage or the presence of hair and beards will decrease efficiency. If you feel unwell, leave the work area and return to areas with fresh air. Avoid use in highly contaminated areas or at risk of explosion.
 - l. The face mask is disposable, it must not be used for more than one work shift.
- a. Risk may be expected if the product is used in a surgical setting or in a setting with significant exposure to bodily or other hazardous liquids or fluids; or used in a clinical setting where the risk of infection through inhalation is high; or used in the presence of a high intensity heat source or flammable gas.
 - b. The product is not designed for use in an anoxic or an underwater environment.
 - c. Check for integrity before use.
 - d. This product is not a respirator.
 - e. The mask does not eliminate the risk of contracting any disease or infection.
 - f. Failure to properly use and maintain this product could result in illness.
 - g. Do not use beyond the period of validity.
 - h. Do not use if the packaging is damaged.
 - i. Do not use with oxygen and only in a ventilated area with > 19% oxygen and for one work shift. Any modifications, damage or the presence of hair and beards will decrease efficiency. If you feel unwell, leave the work area and return to areas with fresh air. Avoid use in highly contaminated areas or at risk of explosion.
 - l. The face mask is disposable, it must not be used for more than one work shift.

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Gima S.p.A.