

INTENDED USE: For use as First Aid Treatment on a burn by Medical Professionals and laypersons alike.

INSTRUCTIONS: Apply a generous layer to affected area. Do not rub into skin. BurnFree® is most effective when applied in a thick layer and allowed to remain in contact with the skin for a minimum of 30 minutes. Re-apply as often as necessary. Cover if necessary.

WARNING: Remove and seek medical advice if rash develops. Seek emergency medical care for severe burns. Device is for external use only. Do not apply compression around gel. Do not use after expiry provided on packaging. Dispose of as clinical waste as per local protocols.

MADE IN USA

MD MEDICAL DEVICE

CE 2797

25°C
15°C
STORAGE LIMIT
TEMPERATURE



 Water-Jel Technologies, LLC
50 Broad Street, Carlstadt, NJ 07072 USA

EC REP  Safeguard Technologies Ltd
Willow Grove, Delgany, Co Wicklow
A63 XY89 Ireland www.burnfree.com

BURN 
FREE 
FIRST AID FOR BURNS

COOLING BURN GEL

FIRST AID FOR BURNS

COOLS • MOISTURIZES
SOOTHES • BURN RELIEF

REF #SD135 REV 10/2021

NET WT. 1/8oz. (3.5g)