GIMA

REF 28606**LOT** 000011

BORSA GHIACCIO diam 28 mm - apertura 5 cm
 ICE BAG diameter 28 mm - small cap 5 cm
 SAC À GLACE diam. 28 mm petite ouverture 5 cm
 BOLSA HIELO diámetro 28 mm - tapa pequeña 5 cm

**UDI**

(01)08023279286069

(11)260712

(10)000011

1 pz.
1 pc.

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WARNING: For external use only. Before every use, always check the wear condition of the device in all its parts, in such a way as to ensure total safe use for people and things. It is forbidden to use the bag otherwise than the one shown. Use only on intact leather. Do not use on skin lesions or tears. Always wear a protective cloth between the skin and the bag. Use only with ice or iced water. In case of use by children, be careful and always keep the bag in check to avoid accidental opening. Before each use check that the surface of the bag does not present any injury, tearing or wear that may affect the liquid seal, as well as the cap seal, checking any leakage. In order to ensure safety, it is recommended that the manufacturer, at least every 2 years, check the suitability for use of the product. For optimal storage of the product, it is recommended to place the bag away from light and from very dry heat sources. To clean the bag both inside and outside, it is recommended to use lukewarm water and a neutral detergent. Always dry in a natural way. Do not expose to sun rays or artificial sources of heat. Keep away from sharp objects that could damage the product. Do not force the lock screw.

CH REP

Medicare Ag
 Hauptstrasse, 51 5024 Kuttigen, Schweiz
 CHRN-AR-20002506

2026-07-12

Rev.3.03.26

Gima S.p.A.
 Via Marconi,1 - 20060 - Gessate (MI) - Italy
 gima@gimaitaly.com - export@gimaitaly.com
 www.gimaitaly.com
 Made in China

**MD**

8 023279 286069

GIMA S.p.A. - Via Marconi, 1 - 20060 Gessate
(MI) - Italy

COD. (TO BE COMPLETED WITH GIMA CODES)

LOT: (TO BE COMPLETED WITH GIMA LOT NR.)

QUANTITY: (TO BE COMPLETED)

MADE IN CHINA

GW: (TO BE COMPLETED)

NW: (TO BE COMPLETED)

CTN NO.: / (TO BE COMPLETED:
FIRST NUMBER CTN NO. / SECOND
NUMBER TOTAL NUMBER OF CAR-
TONS)

SIZE: (TO BE COMPLETED)