Steril gel
Ultrason
20 ml e

ULTRASONİK İLETKEN JEL

İÇERİK: Deionized water, Carbomer, SodiumHydroxide, MonoPropylene Glycol, Phenoxyethanol, Ethylhexylglycerin

ÜRÜN TANIMI

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PRODUCT DESCRIPTION

Sterile contact medium for ultrasound transmission

WARNING

- For medical professional use only
- Do not swallow.
- Stop the procedure in case of hypersensitivity.
- Sterile technique is used to avoid contamination.
- Re-use of the device could result in cross-contamination and infection risk or adverse reaction for the patient.

INTENDED USE

Disposable sterile gel as contact media for invasive medical procedures (diagnostic or therapeutic) where a viscous gel is required.

OPERATION PRINCIPLE

In all ultrasonography and doppler applications, it pushes the air between the skin and the probe, providing better penetration of sonography waves into the device examination area and reducing losses in reflected waves.

APPLICATION

- Apply enough gel on the transducer (probe).
- Follow the instructions of transducer (probe) manufacturer's instructions instead.
- It is recommended to use with a sterile cover for the ultrasound probe / transducer.

- Non-toxic -Not-sensitive -Non-irritating -Non-staining -No formaldehyde -Paraben free -Salt free -Dermatologically tested
- Hypo-allergenic -PVC free -Water soluble -Compatible with PE/PU/ Polyisoprene/Latex covers for ultrasound probes/transducers

UDI
(01) GTINBARCODE
(10) LOTNUMBER
(11) YYMMDD
(17) YYMMDD

LOT 0000000000
REF SUG-0001
YYYY-MM-DD
Y1015.102.0001