


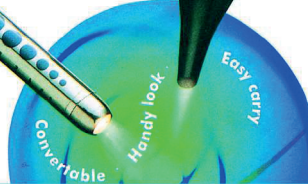

Dual function instrument features convertible penlight and mini-toscope with different sizes of funnel.



- To be used by professional or qualified practitioners only.
- Clip onto the pocket/clothes to prevent accidental discharge of batteries when carried.
- Use only the right type of batteries and bulbs.
- Remove the battery when not in use.

- Place out of the reach of children.
- Don't submerge or soak into water.
- Use only right solution when cleaning.
- Avoid excessive exposure to heat or direct sunlight.
- Keep at room temperature during storage.
- Used batteries and busted lamps must be disposed of properly.

- L'utilizzo è riservato esclusivamente a operatori professionisti o qualificati.
- Fissare mediante clip alla tasca/agli indumenti per evitare che le batterie si scarichino accidentalmente durante il trasporto.
- Usare solo il tipo appropriato di batterie e lampadine.

- Estrarre la batteria quando non è in uso.
- Tenere fuori dalla portata dei bambini.
- Non immergere in acqua.
- Usare solo soluzioni detergenti appropriate.
- Evitare l'esposizione eccessiva al calore o alla luce diretta del sole.
- Conservare a temperatura ambiente.
- Le batterie scariche e le lampade rotte devono essere smaltite correttamente.




Deluxe reusable diagnostic penlight
Light intensity, soft light and touch clips with unique design.
Penna luminosa diagnostica riutilizzabile Deluxe
Intensità luce, luce tenue e clip tattili con design unico.



REF 31300

Gima S.p.A. Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com - www.gimaitaly.com

Rev. 3 - 04/26

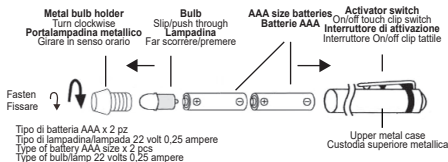[illegible]

CH	REP	Medicare Ag Hauptstrasse, 51 5024 Kuttigen, Schweiz CHRN-AR-20002506
UK	REP	Maclyn Power Ltd 20 Wenlock Road London N1 7GU England, United Kingdom