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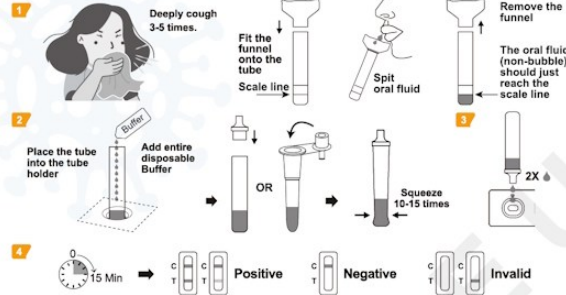
Before Testing
Do not place anything in the mouth including food, drink, gum or tobacco products for at least 10 minutes prior to collection.
Wash your hands with soap and water for at least 20 seconds before and after test. If soap and water are not available, use hand sanitizer with at least 60% alcohol.

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EN COVID-19 Antigen Rapid Test (Oral Fluid) For Self-testing

- Contents:
- Test Device -1
 - Collection Device -1
 - Buffer -1
 - Package Insert -1
 - Biosafety Bag -1

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Do not drink the buffer in the kit. Carefully handle the buffer and avoid it contacting skin or eyes, rinse with plenty of running water immediately if contacting.

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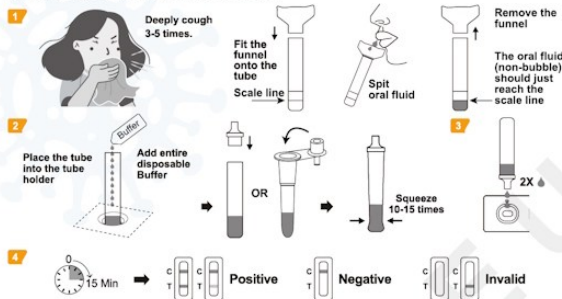
MADE IN CHINA

Read the full package insert carefully before testing.

Before Testing

Do not place anything in the mouth including food, drink, gum or tobacco products for at least 10 minutes prior to collection.
Wash your hands with soap and water for at least 20 seconds before and after test. If soap and water are not available, use hand sanitizer with at least 60% alcohol.

Tube holder

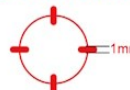
MADE IN CHINA

**COVID-19 Antigen
Rapid Test (Oral Fluid)**

144131004

Φ10.5mm

CE 1434

IVD

For Self-testing | REF: ICQV-802H

ALL TEST

COVID-19 Antigen Rapid Test (Oral Fluid)

COVID-19 Antigen
Rapid Test (Oral Fluid)

EN COVID-19 Antigen Rapid Test (Oral Fluid)

For Self-testing
Contenuto:
• Test Device -1
• Collection Device -1
• Buffer -1
• Package Insert -1
• Biosafety Bag -1

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MedNet GmbH
Hangzhou AllTest Biotech Co., Ltd.