


Neonatal Airway Adapter A3N

It is only applicable to the Capno-T produced by Kingst.
Instruction for use:

1. For monitoring the neonatal patients with endotracheal tube size of 4.0mm or smaller.
2. Press the airway adapter onto the Capno-T, it will 'click' into place when properly seated.
3. Perform Zeroing when replacing new adapter(See manual for details).
4. Install the Capno-T with airway adapter to the ventilator/ anesthetic circuit or oxygen delivery device.
5. DO NOT place the adapter between the endotracheal tube and an elbow to avoid the adapter windows blocked by patient secretions.*

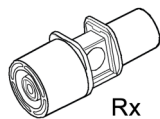

Rx
Only

Single
Patient
Use Only

REF 668504


LOT XXXXXXXX



XXXX-XX



XXXX-XX

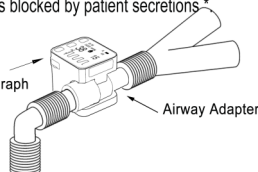

Beijing Kingst Commercial & Trade Co., Ltd.

The 6th floor of No.2-7,Road Beibinhe,
Xibianmen, Dist xicheng,Beijing,P.R.C



SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands



CE 0482


* For more information please see the User Manual

Ver: V1.0

Adattatore per le vie aeree A3N per pazienti pediatrici

È unicamente applicabile al Capno-T prodotto da Kingst.
Istruzioni per l'uso:

1. Per il monitoraggio dei pazienti neonatali con dimensioni del tubo endotracheale minori o uguali a 4,0 mm.
2. Premere l'adattatore per le vie aeree sul Capno-T; un clic segnerà l'inserimento corretto.
3. Quando si effettua la sostituzione con un nuovo adattatore, effettuare l'azzeramento (v. il manuale per i dettagli).
4. Installare il Capno-T con l'adattatore per le vie aeree al ventilatore/circuito anestetico o al dispositivo di erogazione dell'ossigeno.
5. NON posizionare l'adattatore tra il tubo endotracheale e un gomito per evitare che le finestre dell'adattatore siano bloccate dalle secrezioni dei pazienti*.

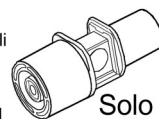

Solo
Rx

Esclusivamente
per uso
monopaziente

REF 668504


LOT XXXXXXXX



XXXX-XX



XXXX-XX

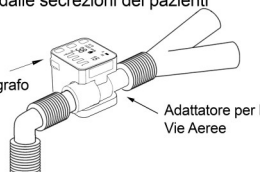


Beijing Kingst Commercial & Trade Co., Ltd.

The 6th floor of No.2-7,Road Beibinhe,
Xibianmen, Dist xicheng,Beijing,P.R.C



SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

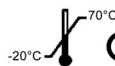

CE 0482

* Per maggiori informazioni, consultare il manuale dell'utente

Ver: V1.0

Importato da / Imported by:

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

MADE IN CHINA

