ABCDEFGHI
J K L M N O P Q R S
T U V W X Y Z
0 1 2 3 4 5 6 7 8 9
* * *

Razormed Inc.
B3 Infocity
Sector 33-34
Gurgaon 122001, India
info@razormed.com
www.razormed.com

Albion Surgicals Limited
Aizlewood's Mill
Nursery Street
Sheffield, S3 8GG, U.K.
Made in India

Razormed[™]

10 Scalpels

Safety Protective Shield Scalpels with Disposal Lock

Stainless Steel Blade

Bistouri de sécurité à enveloppe protectrice avec verrou de blocage. Lame en acier inoxydable
Escudo (funda) protectora de seguridad con cierre desechable. Hoja de acero inoxidable
Bisturi di sicurezza con copri lama autobloccante. Lama in Acciaio Inossidabile
Bisturi de sigurança com escudo protector e dispositivo de bloqueio para descartar. Lâmina em aço inoxidável
Sicherheit Schutzschild Skalpell mit Entsorgungverschluss. Klinge aus Edelstahl
Скальпель из нержавеющей стали
廃棄用のロックを備えた刃に可動性の安全カバーの付いたタイアのセーテスカルペル。ステンレススチール刃
帶可調定位鎖和安全防板的不锈钢刀片外科手术刀

CE 2460
STERILE R
2460
+5°C
+45°C

8063511100

348 04/15

LOT 17516224
2022-11
REF. 27074

20
Fitment No. 4

475
07/17

27074

000006

EN

Not made with natural rubber latex. Single use only. Sterile unless package is opened or damaged. Do not resterilise. Follow safe procedures including disposal of sharps. Used blades are contaminated. Discard safely.

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


DISPOSE LOCK POSITION: To lock scalpel, place thumb on top of button and push forward past the covered position to the locked position. Shield is now locked and scalpel can be safely disposed in an appropriate container.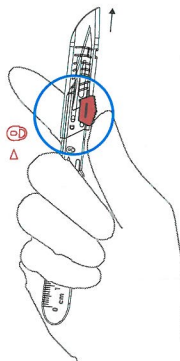

COVERED POSITION: To cover blade, place thumb on top of button and push forward until the blade is fully covered and an audible click is heard to confirm the shield is secured. The scalpel may be reopened and covered again for safety.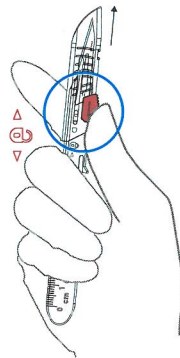

USE POSITION: Retract shield fully until blade is exposed and an audible click is heard to confirm the shield is secured. The scalpel is now ready to be used.

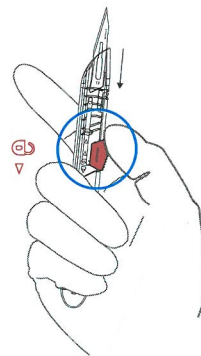OPEN POSITION: To open for use, place thumb on top of button and retract shield back.