

CORE-SCRUB WITH PVP-IODINE 7,5%

Scrub brush/with sponge used for hand cleaning disinfection.

Description

Scrub brush with antiseptic solution, single use, used for hand and forearm disinfection of sanitary personnel. Contains 20 ml antiseptic soap and nail cleaner.

Indications

Device made off a scrub brush, a sponge glued at the base and comes provided with nail cleaner. Used for disinfection of sanitary personnel and surgeons' nail, fingers, hands and forearm, participating in surgery. Exclusively for external use and on healthy and unhurt skin. Used in preoperative areas only

Precaution

Avoid contact with eyes. Do not swallow. Do not use any package with presents sealing. **DO NOT USE ON ANY OTHER PARTS OF THE BODY WHICH ARE NOT INDICATED IN THE PRESENT FILE.**

End-user

Sanitary personnel only.



Warehouse

To be protected from humidity and direct exposure to sunlight, store at max 40°C.

Instruction of use

- Open the individual wrapping, where written "OPEN"
- Distribute the antiseptic solution equally on hands. Use the scrub only for nails and between fingers. Use the sponge for the skin areas.
- Brush and use sponge to disinfect starting from fingers, ending before reaching elbows. Take special care of areas between fingers and nails.
- Clear hands and forearm with running water holding up hands always higher than elbow. The entire procedure lasts 5 minutes.
- Discard scrub brush after use, as by residual protocols and local law application.



TECHNICAL PROPERTIES

Material used	BRUSH: PLD polyethylene SPONGE: Polyurethane - medium high density NAIL CLEANER: PET			
Formula	Components Polivinilpirrolidone-iodine	Concentration % 7,5%	No. CAS 25655-41-8	Risk sentences H412; P501/305/273
Certificates	PMC Ministry of Health Italy: n° 19531 in compliance at EN 528/2012			
Scrub brush dimensions	Lenght: mm 80 Width: mm 50 Height: mm 40			
Nail cleaner dimensions	Lenght: mm 65 Widht: 5 to 10 mm			
Packaging	Boxes of 270 pcs - EWCS270P. 9 dispenser of 30 pieces; total: 270 pieces per outer box			
Pack	Single piece of scrub brush with 20 ml antiseptic solution with nail cleaner, in termoforming pack			
Expiry Date	36 months			

LATEX FREE

Not following the instructions may cause in a improper use of the product. Coremec reserves the right to amplify, modify and to improve the present technical file

COREMEC s.r.l

Sede Legale, Amministrativa e Direzione Commerciale
Corso Vittime del Lavoro, 9 - 10042 Nichelino (TO)
Tel. +39 011-35.89.481 r.a. - Fax +39-011-35.89.483
www.coremec.it e-mail: coremec@coremec.it



COREMEC s.r.l.
Export Department
e-mail: export@coremec.it