**BARELLA DI SOCCORSO
EMERGENCY STRETCHER
BRANCARD D'URGENCE
NOTFALLTRAGE
CAMILLA PARA EMERGENCIAS
MACA DE EMERGÊNCIA
ΦΟΡΕΙΟ ΕΚΤΑΚΤΗΣ ΑΝΑΓΚΗΣ
АВАРИЙНА НОСИЛКА
POHOŤOVOSTNÍ NOSÍTKA
NØDBÅRE
KANDERAAM HÄDAOLUKORRAKS
HÄTÄPAARIT
NOSILA ZA HITNE SLUČAJEVE
VÉSZHELYZETI HORDÁGY
NEŠTUVAI SKUBIAI PAGALBAI
AVĀRIJAS NESTUVES
FØRSTEHJELPSBÅRE
NOODBRANCARD
NOSZE RATUNKOWE
TARGÅ DE URGENTÅ
NÚDZOVÉ NOSIDLÁ
NOSILA ZA NUJNE PRIMERE
NØDBÅR**

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ENGLISH**Name:** Emergency Stretcher
YXH-5E**Accessories:**

two safety belts plastic lock & polypropylene fiber.

Technical data:

1. Product size (LxW xH): 90x51x109cm, Folding size (LxW xH): 114x51x26cm
2. Carton size(1pc): 115x54x27cm
3. N.W.(1pc): 12kg G.W.: 14 kg
4. Load bearing: ≤159kg

Description:

1. The stretcher is mainly used for high building's lift carrying patients. Only need one person to operate this stretcher.
2. It has four wheels and will be easy for moving on ground.
3. The stretcher approves the patient's safety during transport process with two pieces of safety belts.
4. It is made of aluminum alloy materials. It characterized by its being light-weight, small-sized, easy-carried, use-safely and easy for sterilization and clean.

Intended use:

It is used to help the transportation of the injured people.

Attention:

1. When carrying patients with this stretcher, be sure fasten the safety belts for safety sake.
2. In general operation, be careful not to tear out the surface of the stretcher.

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Maintenance:

1. Keep clean as routine.
2. Often check if any parts loose or not.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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